

# CLINICAL

## BONDING PROTOCOLS SUMMARY



### SHIPMENT ARRIVAL:

1. INSPECT TRAYS
  - a. View patients IDB tray images in DIBS AI Software to compare
  - b. Contact DIBS AI Support immediately with any concerns



### 24 HOURS PRIOR TO BONDING APPOINTMENT

1. Apply Adhesive
2. Check if 6's & 7's need extra adhesive



## BONDING APPOINTMENT



### ~ 10 MINUTES PRIOR TO PATIENT'S ARRIVAL

1. Set up tray and include your patients DIBS AI box
2. Open patients IDB tray images from patient profile



### SEATED PATIENT

1. Inspect patients mouth
2. Pumice (whole mouth)
3. Cheek Retractors
4. Etch & Rinse
  - a. Single arch at a time
5. Primer (single tray section)
  - a. Lightly air thin primer
  - b. Light cure 5 slow passes
6. Seat tray
7. Press bracket lightly to tooth
8. Light Cure
  - a. Facial: All teeth in tray section first
  - b. Then alternate curing gingival & incisal per tooth
9. Remove tray before moving to next tray section
10. Repeat steps 5-9



### DEBOND?

1. Remove cement from bracket pad & tooth
2. Bracket reinsertion & DIBS AI tray
  - a. Create tray jig with pin cutter to section, then insert bracket
  - b. Self Ligating Bracket: Close gate prior to bracket being inserted into tray
  - c. Bite Turbos?: Alter tray so bite turbo does not affect tray seating fully
3. Repeat steps 4-9



✉ SUPPORT@DIBSAI.COM  
🕒 +1 801-763-5039  
📞 +1 801-610-1894  
🌐 DIBSAI.COM

